GB

IRUDEK V-230-SLV FFP3 NR

Filtering Half-masks to protect against particles

EN 149:2001

INSTRUCTION FOR USE
"See information supplied by manufacturer"

FITTING INSTRUCTIONS TO BE FOLLOWED EACH TIME RESPIRATOR IN USE

Before use check for visible damage. Damaged or dirty (on breathing side) particle filtering half mask should not be used.

1

Hold the respirator in the hand, allowing the head-straps to hang loosely in front of the mask

2

Position the lower head strap around the neck below the ears. Hold the respirator against the face and place the top head-strap around the crown of the head.

3

Adjust head straps by grasping the loose strap end and pull rear ward until respirator fits comfortably

4

Fit the nose area to the shape of the face by forming the nose piece with fingertips. Start from the nose piece working downward while pressing against the nose. Always use both hands.

If you cannot achieve a proper fit DO NOT enter the contaminated area. See your supervisor.

USE INSTRUCTIONS : The user instruction must be read & followed -

- Failure to follow all instructions and limitations on the use of this respirator and / or failure to wear this respirator during all times of exposure can reduce respirator effectiveness and result in sickness or death.
- Before use wearer must first be trained by the employer for proper respirator use in accordance with applicable Safety and Health Standards. Respiratory protection appliances are to be selected depending on the type and concentration of the hazardous substances.
- Do not use for protection against Gases, Vapor or in atmospheres containing less than 17% Oxygen.
- The respirator may only be used if the type and concentration of the harmful substances are known. In case of unknown substances or concentrations or variable conditions, breathing apparatus should be used.
- Non-ventilated containers, mines, canals should not be entered with the particle filtering half masks & also not allowed in explosive atmosphere.
- If the respirator becomes damaged or breathing becomes difficult, leave the contaminated area, discard and replace the respirator. Also leave the contaminated area immediately if dizziness or other distress occurs.
- Never alter or modify this respirator.
- Do not use with beard or other facial hair that prevent direct contact between the face and edge of the respirator.
- Do not use when concentrations of contaminants are immediately dangerous to life and health, are unknown, or when particulate concentration exceed the maximum use level / or other levels determined by your National Occupational Safety and Health Authorities.

Class EN 149:2001	FFP3 NR
Product Code	IRUDEK-V-230-SLV
Max. Use Level	50 X OEL

OEL : Occupational Exposure Limit

- A multiple use of the particle filtering half mask is limited to one working shift, which means that the mask is designed for the duration of one shift only and must be substituted after the shift.
- NR (Non reusable): A multiple use of the particle filtering half mask is limited to one working shift, which means that the mask is designed for the duration of the shift only and must be substituted after the shift.

USAGE LIFE : Mask gets clogged with dust particles indicated by increase in breathing resistance. The filter respirator is meant for single person use. The duration of use should be maximum single shift.

SHELF LIFE : 60 months from the date of manufacture.
(If stored between -5° C. - +50° C & Humidity not over 80%).

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